# SO ♥ FILL®

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en Instructions For Use

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sv Bruksanvisning

no Bruksanvisning

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**⚠ Warning**

- For safety reasons only SO-FILL syringes should be used for the SO-CONNECT pumps.
- The use of other syringes can damage the SO-CONNECT pump and cause injuries to the patient.
- The manufacturer is not responsible if other than the recommended SO-FILL syringes are used.
- SO-FILL syringes are sterile packed and may not be used if the packaging is damaged.
- The SO-FILL syringes are intended for single use and may not be resterilized.
- For filling the SO-FILL syringe, use only the supplied safety cannula or a withdrawal adapter recommended by the drug manufacturer.
- When filling the syringe, ensure that the volume of the medication filled does not exceed the filling volume of the syringe (pre-programmed syringe size in the pump) or the partial volume programmed into the pump.
- When filling the syringe, ensure that the medication volume filled does not fall below the preprogrammed partial volume. The infusion only starts as intended, when the pump piston and the syringe plunger sit directly on each other.
- To prevent unintended loss of medication during the connection of the SO-FILL syringe with the SO-CONNECT infusion pump, close the reservoir with the Luer Lock cap.
- To minimize the formation of air bubbles, the syringe should be filled and used at room temperature.

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**en** 1. Unscrew and screw the plunger so that it can be unscrewed easily after filling the syringe. • 2. Check the correct "tightening" between the plunger and the piston. • 3. Push the plunger back and forth to uniformly lubricate the silicone inside the body of the SO-FILL syringe. • 4. To fill the SO-FILL syringe, you can use the safety needle or a spike. • 5. Screw the safety needle or the spike on the SO-FILL syringe clockwise. • 6. Fill the SO-FILL syringe by sucking up the liquid slowly. During filling, SO-FILL syringe must be held by the upper part and not by the side. • 7. Verify that the quantity of liquid does not exceed the capacity of the SO-FILL syringe or the defined volume. • 8. If you used the safety needle, after filling the SO-FILL syringe, clip the protective cap to avoid injury. • 9. Unscrew the safety needle or the spike from the SO-FILL syringe counterclockwise. • 10. Unscrew the plunger counterclockwise. • 11. Remove the flange for plunger by turning it counterclockwise. • 12. Turn the head of the pusher on the SO-CONNECT pump • 13. Place the SO-FILL syringe on the SO-CONNECT pump and turn it clockwise.

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